



# Y13560 - Examination Stool Assembly Instructions

## Parts Listing


**A**  
Vinyl Upholstered  
Seat with handle  
1 Unit

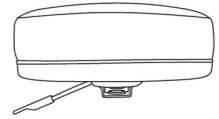


**B**  
Gas Lift  
1 Unit

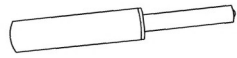

**C**  
Steel Base  
1 Unit

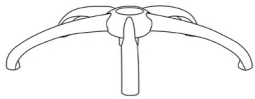

**D**  
PU Casters  
5 Units

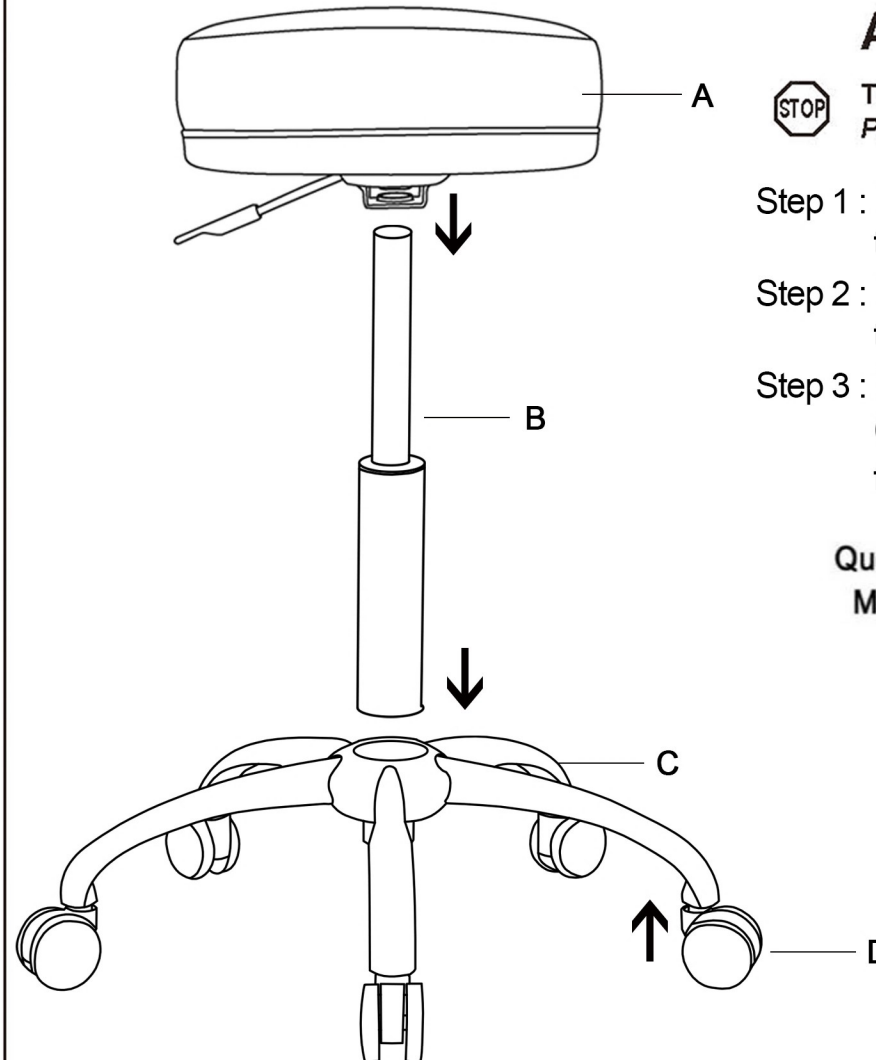

## Assembly Instructions



**Tools Needed:** Phillips Head Screwdriver  
*Please read all instructions before assembly.*

**Step 1 :** Insert the 5 Casters (D) into the outer holes on the Steel Base (C)

**Step 2 :** Insert the bottom end of the Gas Lift (B) into the center hole on the Base (C)

**Step 3 :** Place the Seat (A) on top of the Gas Lift (B)  
(Make sure the top of Gas Lift has been properly fitted into the hole of Seat)

**Questions? Call Customer Service at 1-800-443-5117**  
**Monday - Friday 8:30 AM to 5:00 PM Central Time**

**Weight Capacity : 250 lbs.**

**Assembly Notes:**

During assembly, hand tighten screw only. When all screws are in place, you may then tighten all screws completely.

**CAUTION:**

1. Do not use this chair as a step ladder.
2. Check for loose screws and tighten them every 6 months.